VIA™ enzymatic sponge

Éponge enzymatique VIA™
VIA™ enzymatischer Schwamm
Spugna enzimatica VIA™
Esponja enzimática VIA™
Esponja enzimática VIA™
VIA™ enzymsvamp
Enzymatische spons VIA™
VIA™ enzymatik sünger
VIA™ 효소 스펀지
Губка для ферментной очистки VIA™

Reorder No. 00711687

Référence de commande n° 00711687
Nachbestell-Nr. 00711687
N. di riordino 00711687
N.º de pedido 00711687
N.º de encomenda 00711687
Genbestillingsnr. 00711687
Nabestelnr. 00711687
Yeniden Sipariş Numarası 00711687
주문 번호 00711687
№ повторного заказа: 00711687

INSTRUCTIONS FOR USE

MODE D'EMPLOI
GEBRAUCHSANLEITUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
BRUGSANVISNING
GEBRUIKSAANWIJZING
KULLANIM TALIMATLARI
사용 설명서
ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ

listening... and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

These products have been manufactured not to include natural rubber latex.

Intended Use:

This product is intended to be used for the cleaning of instruments after a procedure.

Warnings and Precautions:

- Review Original Equipment Manufacturer's guidelines regarding the care and cleaning of individual instruments.
- Use appropriate infection control principles and wear appropriate PPE.
- **Never reuse** the sponge or any unused enzymatic as this may lead to cross contamination.
- These devices were not designed to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of these devices to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, reesterilizes, and or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.

Product Description:

- Enzymatic Detergent concentrate:
 - Contains 3 enzymes: Protease, Amylase, Lipase and Surfactant Detergents to break down proteinaceous bioburden during the pre-cleaning of instruments.
 - Low foaming formula.
 - Biodegradable, non-toxic, odorless and non-irritating to the eyes.
- Tube Sponge:
 - Size: 4.25" L x 2.25" D w/0.25" center hole (105mm x 57mm w/6mm center hole).
- Green Friendly, High Density, 100% Medical Grade foam, No Flaking or Scratching. Notched, easy tear open packaging.

Directions for Use:

1. Open package using the tear notch and remove the sponge.
2. Place sponge in a basin or container and activate with 500 ml of water (basin not included).
3. Squeeze the sponge 5-7 times in the water to activate the Enzymatic Detergent solution.
4. Squeeze and remove the sponge from the basin, then use the sponge to wipe down the exterior of the instrument.
5. If applicable, place the distal end of the instrument in the container of freshly made Enzymatic Detergent solution and cycle the fluid through the instrument according to manufacturer's guidelines.
6. Discard the sponge and used Enzymatic Detergent solution using approved methods for materials that have been in contact with possible infectious waste.

Product Disposal:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Packaging is constructed of linear, low-density polyethylene. Please recycle.

Issued Date: December 2015

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions[®], the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.

Manufactured for U.S. Endoscopy by Medtrica in Canada

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Listening...and delivering solutions[®], дизайн US Endoscopy listening...and delivering solutions и все товарные знаки, отмеченные обозначениями © или ™, зарегистрированы в Бюро патентов и торговых знаков США либо являются товарными знаками, принадлежащими компании US Endoscopy.
Listening...and delivering solutions[®], le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe [®] ou [™] sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions[®], das US Endoscopy ...and delivering solutions-Design und alle mit [®] oder [™] gekennzeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions[®], o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions[®], designet US Endoscopy listening...and delivering solutions og alle varemærker markeret med ® eller ™ er registreret i USA's patent- og varemærkereregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 USEndoscopy 가 소유하고 있는 상표입니다.