Ultramatrix™ EUS balloon - Olympus/Pentax radial
Ballonnet de distension EUS Ultramatrix™ - Olympus/Pentax radial
Ultramatrix™ EUS-Ballon – Olympus/Pentax radial
Ultramatrix™ Palloncino EUS - Olympus/Pentax radiale
Balón de ecoendoscopia Ultramatrix™ - Olympus/Pentax radial
Balão EUS Ultramatrix™ - Olympus/Pentax radial
Ultramatrix™ EUS-ballon – Olympus/Pentax radial
Ultramatrix™ EUS-ballon - Olympus/Pentax radiaal
Ultramatrix™ EUS balon - Olympus/Pentax radyal
Ultramatrix™ EUS 벌룬 - Olympus/Pentax 방사형
Баллон для ЭУИ Ultramatrix™ — радиальный Olympus/Pentax
Ultramatrix™ EUSバルーン - Olympus/Pentax ラジアル

Reorder No. USBOR
Référence de réapprovisionnement USBOR
Bestell-Nr. USBOR
N. di riordino USBOR
N.º de pedido USBOR
N.º de encomenda USBOR
Genbestillingsnr. USBOR
Nabestelnr. USBOR
Yeniden Sipariş No. USBOR
주문 번호 USBOR
№ для повторного заказа USBOR
再注文番号 USBOR

INSTRUCTIONS FOR USE

MODE D'EMPLOI

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BRUGSANVISNING

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KULLANIM TALİMATLARI

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*listening... and delivering solutions**

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

Not made with natural rubber latex.

Intended Use:

The US Endoscopy Ultramatrix™ EUS balloon is intended to be used for endoscopic ultrasound applications using the balloon method in the upper or lower gastrointestinal tract.

Warnings and Precautions:

- Consult the medical literature relative to techniques, complications and hazards prior to the performance of any EUS procedure.
- Inspect the balloon prior to use to ensure it has no holes, swelling, color changes or other irregularities. If an irregularity is detected, **DO NOT USE THE BALLOON**. Obtain another balloon and **contact your local Product Specialist**.
- The balloon can easily tear, so beware of sharp objects.
- Do not apply excessive force to or bend the distal end of the echoendoscope. Instrument damage may occur.
- Do not feed water into the balloon when the water level in the water container is too low. The balloon may not inflate properly.
- Do not disconnect the water container from the endoscope just after removing air bubbles from the balloon. This may cause the air bubbles to re-enter the balloon when attempting to inflate the balloon again.
- If using a thread to secure balloon, use caution to prevent occlusion of the balloon water supply/suctions ports of the echoendoscope.
- The EUS Balloon may burst or come unseated if inflated to a diameter greater than the recommended 20 mm.
- If balloon doesn't deflate for echoendoscope extubation, and removal, please see directions below.
- Do not attempt to reuse, reprocess, refurbish, remanufacture or resterilize this device. US Endoscopy did not design this device nor is it intended to be reused, reprocessed, refurbished, remanufactured, or reesterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e. compromised device integrity, cross-contamination, infection).
- EUS procedures should only be performed by persons that have adequate training and familiarity with EUS techniques.

Contraindications:

- Contraindications include those specific to any endoscopic procedure, as well as those specific to endoscopic ultrasound procedures.

Directions for Use:

1. Prior to clinical use, familiarize yourself with the device and read all the Instructions for Use.

Section 1 - Attaching the balloon:

Applying balloon with Olympus balloon applicator (MAJ-864):

1. Prime balloon water supply channel by means recommended by echoendoscope manufacturer.
2. Stretch balloon three times by gently pulling the two ends of the balloon apart and allowing them to retract.
3. Insert front end of balloon into applicator and stretch the rear band of the balloon and insert it into the attachment groove.
4. Insert the distal tip of the echoendoscope into the balloon (and applicator) as far as possible.
5. Roll the rear band of balloon off the applicator groove and into the proximal balloon attachment groove of echoendoscope.
6. Gently attach the front band of the balloon to the scope's distal attachment groove.
7. Tug on the balloon circumferentially between the front and rear bands to help them better seat within the attachment grooves.
8. If using a thread to secure balloon, tie it within the attachment grooves of the echoendoscope.

Applying the balloon without applicator:

1. Follow instructions one and two from section above discussing the use of the applicator for balloon attachment.
2. Stretch open the rear band of the balloon slightly to allow for placement onto transducer tip.
3. Insert the tip of the echoendoscope into the balloon by alternately stretching and pulling balloon proximally.
4. Roll the rear band into the proximal balloon attachment groove of the echoendoscope once tip is fully inserted.
5. Gently attach the front band of the balloon to the scope's attachment groove.
6. Pull on the balloon circumferentially between the front and rear bands to help them better seat within the attachment grooves.
7. If using a thread to secure balloon, tie it within the attachment grooves of the echoendoscope.

Section 2 - Priming the balloon and expelling air:

1. With the end of the insertion tube pointed down, depress the air/water valve completely and fill the balloon with de-aerated water.
2. Inflate the balloon to approximately 20 mm, relax finger depressing air/water valve and make sure balloon is inflated.
3. Confirm that there is no evidence of any air bubbles within the balloon.
4. If no air bubbles are present, depress the suction valve completely, removing all the water from the balloon.
5. If air bubbles are present within the balloon, repeat steps 1 through 4 until all the air bubbles are completely aspirated.
6. Now the balloon is ready to use.

Section 3 - Removing balloon from echoendoscope:

1. While watching ultrasound image, deflate balloon by pressing down on the suction button completely and aspirating all the water.
 - a. If balloon doesn't deflate proceed with the following steps:
 - i. Turn airflow/CO2 regulator off.
 - ii. Remove the air/water valve from the echoendoscope.
 - iii. Lower control head of echoendoscope below patient's bed.
 - iv. Visualize balloon deflation on ultrasound screen.
 - v. Replace air/water valve.
2. Completely depress suction valve while removing echoendoscope from patient.
3. After removing echoendoscope from the patient, carefully detach balloon from transducer.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: February 2018

Warning:

An issued or revision date for these instructions is included for the user's information. In the event two years has elapsed between this date and product use, the user should contact **US Endoscopy** to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.

Olympus® is a registered trademark of Olympus Optical Co., Ltd.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das „listening...and delivering solutions“-Design und alle mit ® oder ™ versehenen Marken sind beim United States Patent and Trademark Office (Patentamt der Vereinigten Staaten) registriert oder befinden sich im Besitz von US Endoscopy
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy
Listening...and delivery solutions®, designed af US Endoscopy listening...and delivery solutions, og alle mærker markeret med ® eller ™ er registreret hos U.S. Patent and Trademark Office eller varemærker ejet af US Endoscopy
Listening...and delivering solutions[®], het ontwerp Listening...and delivering solutions van US Endoscopy en alle aanduidingen met [®] of [™] zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 US Endoscopy가 소유하고 있는 상표입니다.