



Baleen™ polyp trap

quick reference guide


NBPT520

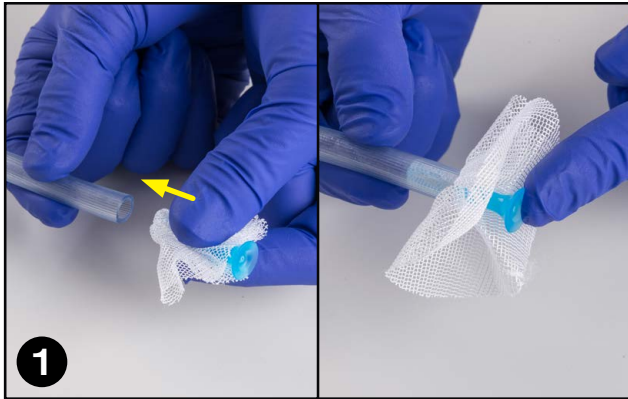

Insert the narrow tip end of the Baleen™ polyp trap applicator with netting into the suction tubing until you meet resistance.

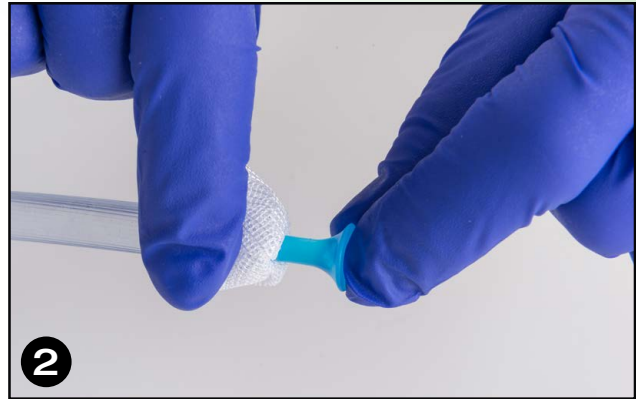

Fold excess netting back over the end of the suction tubing and hold securely.

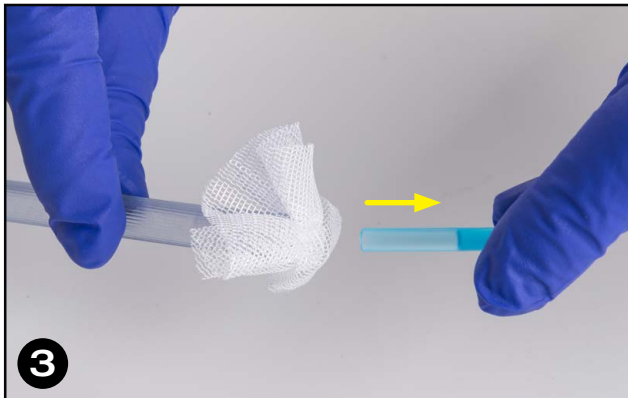

Grab flange-end of the Baleen™ polyp trap applicator and remove from the suction tubing, leaving the netting in place. Discard the applicator.



Attach the suction tubing with the intact Baleen™ polyp trap to the suction connector (barb) of endoscope. The Baleen™ polyp trap is now in place and ready for use.



After the polyp has been suctioned into the Baleen™ polyp trap, disconnect the suction tubing from umbilicus of the endoscope (holding onto the netting). To remove the polyp, grasp all exposed netting and pull the netting out of the suction tubing.

us endoscopy
listening...and delivering solutions®

This information is to be used as a reference guide only. Review the instructions for use prior to using the device.

US Endoscopy is a subsidiary of STERIS Corporation.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with © or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

Copyright© 2018 US Endoscopy and affiliates. All rights reserved.

761480 Rev. A