



GENERAL DESCRIPTION

Revital-Ox Bedside Concentrate is an enzymatic detergent and infused sponge formulated to pre-clean and remove protein, carbohydrates and lipid soils from the exterior and interior of flexible or rigid endoscopes and surgical instruments.

FEATURES

Concentrated
Low foaming
Individual package
Flat sponge
Large surface area

BENEFITS

Fresh enzymatic solution with each pre-cleaning process
Easy to rinse
One-time use
Easily contours to the shape of any device; easy to store slim pouch
Minimizes staff's direct contact with device

TYPICAL PHYSICAL PROPERTIES

Form.....Flat sponge with enzymatic solution
Odor.....No fragrance added
Color (Sponge).....Blue

DIRECTIONS FOR USE

For Endoscope Application:

Tear open the package at the notch. Add warm tap water to bring volume to the designated fill line. While still in the pouch, lightly massage the sponge to mix the enzymatic concentrate with the water. Squeeze excess enzymatic detergent from the sponge. Remove the sponge from the pouch and wipe down the outside of the endoscope. Place the distal end of the endoscope into the pouch and suction the remaining enzymatic detergent through the biopsy suction channel. Cycle air and detergent through the channel. Purge the channel with clear water and rinse the endoscope thoroughly. Place the endoscope into the transport container and send to the reprocessing area. Keep the endoscope moist until final cleaning.

For Surgical Instrument Application:

Tear open the package at the notch. Add warm tap water to bring volume to the designated fill line. While still in the pouch, lightly massage the sponge to mix the enzymatic concentrate with the water. Squeeze excess enzymatic detergent from the sponge. Remove the sponge from the pouch and wipe down the outside of the device. Rinse thoroughly.

Discard packaging and sponge immediately after use – for single use only.

STORAGE AND DISPOSAL

Store in a cool place (below 90°F [32°C]). Keep out of reach of children. Dispose in accordance with local, state, and federal regulations. Empty containers should be rinsed and discarded in a waste receptacle or offered for recycling if possible. Do not use empty containers for other purposes.

SUPPORT

Service

Service is one of the most important ways to verify consistent quality of the facility's performance and operation. A tailored service program by STERIS provides effective, trouble-free operations.

Technical

Contact STERIS at the 800-number listed below for information regarding product applications, the product in general or copies of Safety Data Sheet (SDS). In case of emergency, call STERIS's St. Louis Operations regulatory affairs group at 314-535-1395.

For further information, please contact:

STERIS[®]



STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 • USA
440-354-2600 • 800-548-4873
www.steris.com