BioShield® – biopsy valve – sterile
BioShield® – valve à biopsie – stérile
BioShield® – Biopsieventil – steril
BioShield® – valvola bioptica – sterile
BioShield® – válvula de biopsia – estéril
BioShield® – válvula de biopsia – estéril
BioShield® – biopsiventil – steril
BioShield® – klep voor biopsie – steriel
BioShield® – biyopsi kapağı – steril
BioShield® – 용종판막 – 멸균형

Reorder No. 00711128
Référence de commande 00711128
Nachbestell-Nr. 00711128
N. di riordino 00711128
N.º de pedido 00711128
N.º de encomenda 00711128
Genbestillingsnr. 00711128
Nabestelnr. 00711128
Yeni Sipariş Numarası: 00711128
주문 번호 00711128

INSTRUCTIONS FOR USE
MODE D'EMPLOI
GEBRAUCHSANLEITUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
BRUGSANVISNING
GEBRUIKSAANWIJZING
KULLANIM TALİMATLARI
사용 설명서

listening... and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

STERILE EO

RX Only (U.S.A.)

INTENDED USE:

The BioShield® biopsy valve – sterile is used to cover the opening to the biopsy/suction channel inlet of a Olympus® and Fujinon® (G5 series and newer) gastrointestinal endoscope. The BioShield® biopsy valve – sterile provides access for endoscopic device passage and exchange, helps maintain insufflation and minimizes leakage of biomaterial from the biopsy port throughout the gastrointestinal endoscopic procedure.

WARNINGS AND PRECAUTIONS:

- When irrigation is used, ensure proper techniques to avoid aspiration in the patient.
- Do not use a sharp or pointed object to prime the biopsy valve prior to use.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.
- Do not leave a device hanging from the valve. Doing so can cause the creation of larger valve slit / hole that can compromise leak management.
- If the lid of the BioShield® biopsy valve - sterile is opened while attached to the endoscope during a procedure, scope suction will be compromised and chances of leakage increase. A sterile gauze should be used to cover the biopsy valve if the lid must be opened for any reason.
- Exposure to bodily fluids may occur during connection or disconnection of these devices; adherence to Body Substance Isolation protocols is the responsibility of the user.

DIRECTIONS FOR USE (BY PRODUCT NUMBER):

1. Securely place the biopsy valve onto the biopsy/suction channel opening of Olympus (excluding Olympus linear EUS endoscopes) or G5 series and newer Fujinon gastrointestinal endoscope.
2. This biopsy valve does not accommodate a luer-lock syringe for irrigation purposes without an irrigating adaptor such as US Endoscopy part #00711131 (sold separately).

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: October 2010

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with © or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

Olympus® is a registered trademark of Olympus Optical Co., Ltd.

Fujinon® is a registered trademark of Fuji Film Co., Ltd.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
„Listening...and delivering solutions®“, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, il design dello slogan listening...and delivering solutions di US Endoscopy e tutti i marchi contrassegnati da ® o ™ sono stati depositati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, designet US Endoscopy listening...and delivering solutions og alle varemærker markeret med ® eller ™ er registreret i USA's patent- og varemærkeregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, het ontwerp listening...and delivering solutions van US Endoscopy en alle aanduidingen met ® of ™ zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.