Torrent[®] irrigation scope connector
Connecteur pour endoscope d'irrigation Torrent[®]
Torrent[®] Endoskop-Spülanschluss
Connettore dello scopio per irrigazione Torrent[®]
Conector de endoscopio de irrigación Torrent[®]
Conector de endoscópio Torrent[®]
Torrent[®] udskylningstilslutning
Torrent[®] irrigatiescoopconnector
Torrent[®] irigasyon alanı konnektörü
Torrent[®] 세정 스코프 커넥터
Ирригационно-эндоскопный переходник Torrent[®]

Reorder No. 00711570
Référence de commande 00711570
Nachbestell-Nr. 00711570
N. di riordino 00711570
N.º de pedido 00711570
N.º de encomenda 00711570
Genbestillingsnr. 00711570
Nabestelnr. 00711570
Yeni Sipariş No. 00711570
주문 번호 00711570
№ для заказа 00711570

INSTRUCTIONS FOR USE
MODE D'EMPLOI
GEBRAUCHSANLEITUNG
ISTRUZIONI PER L'USO
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
BRUGSANVISNING
GEbruIKSAANWIJZING
KULLANIM TALİMATLARI
사용 설명서
ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ

*listening...and delivering solutions**

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

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This product has been manufactured not to include latex.

INTENDED USE:

The Torrent® irrigation scope connector is used in conjunction with the Torrent® irrigation tubing (tubing and accessories to accommodate various endoscopes and irrigation pumps) and are intended to provide irrigation via irrigation fluids such as sterile water supplied during gastrointestinal endoscopic procedures when used in conjunction with an irrigation pump (or electrosurgical unit).

For use with Olympus endoscopes series 140, 160, 180 and 190, specifically those endoscopes that have an auxiliary water port.

WARNINGS AND PRECAUTIONS:

- **This disposable medical device is intended for single patient use only.**
- Never attach or reattach the Torrent® irrigation scope connector to an endoscope that is/was inside the patient and has not been reprocessed.
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- This disposable medical device is not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses this disposable medical device must bear full responsibility for their safety and effectiveness.
- Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
- Consult the medical literature relative to complications, hazards, and techniques prior to the performance of any endoscopic procedure.
- Use proper aseptic techniques to avoid contamination during device setup.
- Exposure to bodily fluids may occur during the connection or disconnection of this device. The User is responsible for adhering to Body Substance Isolation protocols.
- Do not insert this device into the patient. The device should be attached to the auxiliary water port of a flexible gastrointestinal endoscope and an irrigation source. Use proper techniques to avoid risk of aspiration of a patient.
- Always prime the auxiliary water channel or biopsy channel prior to inserting the endoscope into the patient.

CONTRAINDICATIONS:

Contraindications include those specific to any endoscopic procedure.

INSTRUCTIONS FOR USE:

1. Prior to clinical use familiarize yourself with the device.
2. Open the sterile pouch and attach the Torrent® irrigation scope connector to the Torrent® irrigation tubing (00711560 / 00711561 / 00711562 / 00711563). For use of the Torrent® irrigation tubing, reference Instructions for Use.
3. Attach the Torrent® irrigation scope connector to the auxiliary water port by screwing it onto the port. **When attaching the scope connector to the port, take care to ensure that the scope connector is aligned to the port and avoid excessive tightening.** Misalignment or excessive tightening of the scope connector could lead to the possibility of leakage. Once the scope connector is attached to the auxiliary water port of the endoscope, attach Torrent® irrigation tubing to the Torrent® irrigation scope connector. The Torrent® irrigation scope connector is manufactured with a one-way valve to minimize the risk of cross-contamination of the Torrent® irrigation system.
4. **Discard the Torrent® irrigation scope connector after each patient use.**

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: March 2013

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

Olympus® is a registered trademark of Olympus Optical Co., Ltd.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le symbole ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, il design dello slogan listening...and delivering solutions di US Endoscopy e tutti i marchi contrassegnati da ® o ™ sono stati registrati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivery solutions®, US Endoscopy listening...and delivery solutions-designet og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 US Endoscopy 가 소유하고 있는 상표입니다.