**Keeper™ device clip**  
**Pince Keeper™**  
**Keeper™ Geräteklemme**  
**Clip del dispositivo Keeper™**  
**Pinza para dispositivo Keeper™**  
**Grampo de dispositivo Keeper™**  
**Keeper™ -enhedsklemme**  
**Keeper™ hulpclip**  
**Keeper™ cihaz mandalı**  
**Keeper™ 장치 클립**  
**Зажим для свернутых катетеров Keeper™**

Reorder No. 00711751, 00711752  
Référence de commande 00711751, 00711752  
Nachbestell-Nr. 00711751, 00711752  
N. di riordino 00711151, 00711752  
Nº de pedido 00711751, 00711752  
N.º de encomenda 00711751, 00711752  
Genbestillingsnr. 00711751, 00711752  
Nabestelnr. 00711751, 00711752  
Yeni Sipariş No: 00711751, 00711752  
주문 번호 00711751, 00711752  
№ для заказа: 00711751, 00711752

---

**INSTRUCTIONS FOR USE**  
**MODE D'EMPLOI**  
**GEBRAUCHSANLEITUNG**  
**ISTRUZIONI PER L'USO**  
**INSTRUCCIONES DE USO**  
**INSTRUÇÕES DE UTILIZAÇÃO**  
**BRUGSANVISNING**  
**GEBRUIKSAANWIJZING**  
**KULLANIM TALİMATLARI**  
**사용 설명서**  
**ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ**

---

*listening...and delivering solutions\**

5976 Heisley Road  
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email [global@usendoscopy.com](mailto:global@usendoscopy.com)

[www.usendoscopy.com](http://www.usendoscopy.com)

**NON-STERILE**

***This device is not made with natural rubber latex.***

**Intended Use:**

The Keeper™ device clip is a component designed to be used for holding/managing coiled, catheter type accessory devices that are used during endoscopic procedures.

**Warnings and Precautions:**

- The Keeper™ device clip was not designed to be reprocessed or reused, and therefore US Endoscopy cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure effective performance.
- The Keeper™ device clip product line is intended for single patient use only. Any institution, practitioner, or third party, who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses these disposable devices must bear full responsibility for their effectiveness.

**Contraindications:** Contraindications include those specific to any endoscopic procedure.

**DIRECTIONS FOR USE:**

1. Prior to clinical use, familiarize yourself with the Keeper™ device clip, and read all the Instructions for Use.
2. Open the pouch and remove the Keeper™ device clip.
3. Coil the intended accessory device loosely.
4. Open the Keeper™ device clip by squeezing the handle.
5. Place the accessory device in the opened jaw of the Keeper™ device clip.
6. Close the Keeper™ device clip by releasing the handle.
7. Place clipped accessory device on flat surface or within a secured pocket until needed.
8. When clipped accessory device is needed, squeeze the handle of the Keeper™ device clip and remove the accessory device.
9. Uncoil the accessory device for use.
10. Discard the Keeper™ device clip after the procedure is completed.

**Product Disposal:**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

**Issued Date:** March 2012

**Warning:**

An issued or revision date for these instructions is included for the user's information. In the event two years has elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening... and delivering solutions®, the US Endoscopy listening... and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le sigle® ou™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind eingetragene Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions® , il design dello slogan listening...and delivering solutions di US Endoscopy e tutti i marchi contrassegnati da ® o ™ sono stati registrati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos E.U.A. (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions-designet og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.