BioGuard® air/water cleaning adapter – Olympus

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BRUGSANVISNING

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KULLANIM TALİMATLARI

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listening... and delivering solutions®

5976 Heisley Road
Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

CE

Not made with natural latex rubber.

Intended use: This device is intended as part of the precleaning process, to help clear the air/water channel of Olympus GI endoscopes post procedure.

Device description: BioGuard® air/water cleaning adapter is a valve that fits onto the air/water cylinder of Olympus endoscopes (that do not contain a balloon channel). It is used post procedure, during the precleaning stage of the endoscope cleaning process. This device flushes the air/water channel with water and air to keep the air/water channel from clogging with debris.

Warnings and Precautions:

- **These devices are intended to be used with Olympus® GI Endoscopes **EXCLUDING Olympus® echoendoscopes that utilize a balloon channel****
- **Do Not Use** the BioGuard® air/water cleaning adapter during patient procedures. Continuous air flow could cause patient injury.
- Endoscopic cleaning procedures should only be performed by persons having adequate training with endoscope cleaning techniques.
- Do not attempt to reuse, reprocess, refurbish, remanufacture or resterilize this device. US Endoscopy did not design this device, nor is it intended to be reused, reprocessed, refurbished, remanufactured, or resterilized. Performing such activities on this disposable medical device presents a safety risk to patients (i.e. compromised device integrity, cross-contamination, infection).

Contraindications:

- Contraindications include using on GI endoscopes that are not Olympus or the Olympus echoendoscope that has a balloon channel.
- Use during patient procedures in place of standard air/water valve for insufflation and lens washing.

Prior to Use:

1. Prior to clinical use, inspect and familiarize yourself with the device. If there is evidence of damage (i.e. deformed, cracked/broken, damaged packaging), **do not use this product, save for return, and contact your local Product Specialist.**

Directions for Use:

1. Remove the device from the package.
2. Turn off the airflow regulator on the light source after initial rinsing of scope (per facility guidelines).
3. Remove the air/water valve from the scope.
4. Insert the BioGuard® air/water cleaning adapter into the air/water cylinder on scope handle.
5. Place the distal end of the scope into a basin of water.
6. On the light source, turn on the airflow regulator and set it to a maximum output of "high".
7. Depress the BioGuard® air/water cleaning adapter for the recommended time per the scope manufacturer's reprocessing instructions.
Note: This flushes the air/water channel with water.
8. Release the BioGuard® air/water cleaning adapter.
9. Keep the distal end of the scope in the basin of water.
10. Flush air for the recommended time per the scope manufacturer's reprocessing instructions.
Note: Air bubbles will be seen in the basin of water.
11. Turn off the air/flow regulator on the light source.
12. Remove distal end of scope from basin of water.
13. Remove the BioGuard® air/water cleaning adapter from scope after precleaning step is completed.
14. Discard the BioGuard® air/water cleaning adapter in accordance with institutional guidelines.

**Product Disposal:**

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: October 2018

Warning:

An issued or revision date for these instructions is included for the user's information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions[®], the US Endoscopy listening...and delivering solutions design, and all marks denoted with [®] or [™] are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy

US Endoscopy, a wholly owned subsidiary of STERIS Corporation.

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Listening...and delivering solutions[®], le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe [®] ou [™] sont des marques déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions[®], das „listening...and delivering solutions“-Design und alle mit [®] oder [™] versehenen Marken sind beim United States Patent and Trademark Office (Patentamt der Vereinigten Staaten) registriert oder befinden sich im Besitz von US Endoscopy.
Listening...and delivering solutions[®], o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com [®] ou [™] estão registadas no Departamento de Registos de Patentes dos EUA (U.S. Patent and Trademark Office) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions[®], designet af US Endoscopy listening...and delivering solutions, og alle mærker markeret med [®] eller [™] er registreret hos U.S. Patent and Trademark Office eller varemærker ejet af US Endoscopy.
Listening...and delivering solutions[®], het ontwerp Listening...and delivering solutions van US Endoscopy en alle aanduidingen met [®] of [™] zijn gedeponeerd bij het Patent and Trademark Office in de Verenigde Staten of zijn handelsmerken van US Endoscopy.
Listening...and delivering solutions[®], US Endoscopy listening...and delivering solutions 디자인, 그리고 [®] 또는 [™]으로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 US Endoscopy 가 소유하고 있는 상표입니다.