BioShield® – irrigating adaptor

BioShield® – adaptateur pour irrigation
 BioShield® – Spüladapter
 BioShield® – adattatore per irrigazione
 BioShield® – adaptador de irrigación
 BioShield® – adaptador de irrigação
 BioShield® – adapter til skylning
 BioShield® – irrigatieadapter
 BioShield® – irigasyon adaptörü
 BioShield® – 세정 어댑터
 BioShield® – ирригационный адаптер

BioShield® irrigator

Irrigateur BioShield®
 BioShield® – Irrigator
 BioShield® – irrigatore
 BioShield® – irrigador
 BioShield® – dispositivo de irrigação
 BioShield® – udskylningsapparat
 BioShield® – irrigator
 BioShield® – irigatör
 BioShield® 세정기
 BioShield® – ирригатор

BioShield® irrigator – extension tubing (180cm)

Irrigateur BioShield® – tube de rallonge (180 cm)
 BioShield® Irrigator – Verlängerungsschlauch (180 cm)
 BioShield® – tubo di prolunga per irrigatore (180 cm)
 BioShield® – irrigador – tubo de extensión (180 cm)
 BioShield® – dispositivo de irrigação – tubo de extensão (180 cm)
 BioShield® – udskylningsapparat – forlængerslange (180 cm)
 BioShield® – irrigator – verlengbuis (180 cm)
 BioShield® – irigatör – uzatma tüpü (180cm)
 BioShield® – 세정기 – 기존 튜브(180cm)
 BioShield® – удлинитель ирригатора (180 см)

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listening... and delivering solutions™

5976 Heisley Road
 Mentor, OH 44060 USA

phone +1 440 / 639.4494

fax +1 440 / 639.4495

email global@usendoscopy.com

www.usendoscopy.com

This product has been manufactured not to include latex.

INTENDED USE:

The BioShield® biopsy valve is used to cover the opening to the biopsy/suction channel inlet of a gastrointestinal endoscope. The BioShield® biopsy valve provides access for endoscopic device passage and exchange, helps maintain insufflation and minimizes leakage of biomaterial from the biopsy port throughout the gastrointestinal endoscopic procedure.

A variety of (8) different BioShield® biopsy valves are available. BioShield® biopsy valve (part #00711125, #00711124, #00711126, #00711129, #00711133, #00711135) is used to cover the opening to the biopsy/suction channel of Olympus (excluding Olympus linear EUS endoscopes) and G5 series and newer Fujinon gastrointestinal endoscopes only. BioShield® biopsy valve (part #00711127, #00711136, #00711137) is used to cover the biopsy/suction channel of Pentax gastrointestinal endoscopes only.

The BioShield® irrigator (part #00711133), appropriate for use with Olympus and G5 series or newer Fujinon gastrointestinal endoscopes, and the BioShield® irrigator (part #00711137), appropriate for use with Pentax gastrointestinal endoscopes, will also allow for remote irrigation via an attached irrigation line.

The BioShield® irrigating adaptor (part #00711131) is used for intraprocedural gastrointestinal endoscopic irrigation when attached to a luer-lock or slip-tip syringe through the BioShield® biopsy valve.

The BioShield® irrigator extension tubing (part #00711134) is used for attachment to an auxiliary water pump for hands-free foot pedal irrigation control.

WARNINGS AND PRECAUTIONS:

- The BioShield® biopsy valve part # selected by the clinician must accommodate the type/manufacturer of gastrointestinal endoscope utilized. Selection is determined in accordance for use with Olympus (excluding Olympus linear EUS endoscopes) and G5 series and newer Fujinon gastrointestinal endoscopes, or for use with Pentax gastrointestinal endoscopes.
- Syringe is not included.
- Closing the pinch clamp attached to the irrigation line on part #00711133 and part #00711137 prevents “whistling” when the irrigation line is not in use.
- Irrigating through the BioShield® biopsy valve’s irrigation line with a device in the biopsy channel requires sufficient space between the diameter of the device and the diameter of the endoscope’s channel. Adequate space is required to allow fluid to pass freely at a moderate pressure so that the valve connections do not leak.
 - ◇ **Example: 2.3mm device in a 2.8mm channel (space .5mm) = inadequate space to allow for moderate pressure delivery of fluids; leakage may occur.**
 - ◇ **Example: 2.3mm device in a 3.2mm channel (space .9mm) = space is increased, keeping pressure in the moderate range, thus limiting the potential for valve connection leakage.**
- **FOR PART #00711133 AND #00711137: Leakage through non-sheathed spring-coiled devices such as biopsy forceps will occur if irrigation is performed while the non-sheathed device remains in the biopsy channel.**
- US Endoscopy did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- These disposable medical devices are not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses these disposable medical devices must bear full responsibility for their safety and effectiveness.
- When irrigation is used, ensure proper techniques to avoid aspiration in the patient.
- Exposure to bodily fluids may occur during connection or disconnection of these devices; adherence to Body Substance Isolation protocols is the responsibility of the user.
- Do not use a sharp or pointed object to prime the valve prior to use.
- Do not leave a device hanging from the valve. Doing so can cause the creation of a larger valve slit/hole that may cause leakage.
- If the lid of the valve is opened while attached to the endoscope during a procedure, scope suction will be compromised and leakage may occur.
- If leakage occurs, a sterile gauze should be used to cover the valve.

FOR PART #00711133, #00711134 AND #00711137: Product contains Phthalates

DIRECTIONS FOR USE (BY PRODUCT NUMBER):

00711131 BioShield® – irrigating adaptor

1. Attach the irrigating adaptor to luer-lock or slip tip syringe; fill syringe with institutionally approved irrigation fluid (eg. sterile water).
2. Syringe with irrigating adaptor may now be used as needed for intraprocedural irrigation by gently placing the assembled syringe and irrigating adaptor into the biopsy valve and depressing the syringe plunger until desired irrigation effect is achieved. Repeat as needed.

00711133 BioShield® irrigator

1. Securely place the biopsy valve with irrigation line onto the biopsy/suction channel opening of Olympus (excluding Olympus linear EUS endoscopes) or G5 series and newer Fujinon gastrointestinal endoscope.
2. Close the pinch clamp attached to the irrigation line to prevent “whistling” until attachment of a syringe or auxiliary water pump extension tubing for the purpose of irrigating is desired.
3. Fill luer-lock syringe with institutionally approved irrigation fluid (eg. sterile water).
4. To irrigate, open the pinch clamp on the irrigation line and attach the fluid-filled syringe to the irrigation line; depress syringe plunger until desired irrigation effect is achieved; repeat as necessary. **ALTERNATE METHOD:** open the pinch clamp on the irrigation line; attach part #00711134 BioShield® irrigator extension tubing (180cm) to compatible auxiliary water pump, securing the male end of the tubing to the BioShield® device’s irrigation line. Depress the water pump’s foot pedal until desired irrigation effect is achieved; repeat as necessary.
5. Close the pinch clamp on the irrigation line when not in use for irrigation; this will prevent “whistling.”

00711134 BioShield® irrigator – extension tubing (180cm)

1. Refer to endoscope manufacturers’ instruction booklet for maximum PSI setting.
2. Securely attach the female end of the extension tubing to an auxiliary water pump and the male end to the BioShield® device’s (part #00711133 or #00711137) irrigation line.
3. Irrigate as needed or until desired irrigation effect is achieved.

00711125 / 00711124 / 00711126 / 00711129 / 00711135 BioShield® – biopsy valve

1. Securely place the biopsy valve onto the biopsy/suction channel opening of Olympus (excluding Olympus linear EUS endoscopes) or G5 series and newer Fujinon gastrointestinal endoscope.
2. This biopsy valve does not accommodate a luer-lock syringe without an irrigating adaptor (part #00711131; sold separately).

00711127 / 00711136 BioShield® – biopsy valve

1. Securely place the biopsy valve onto the biopsy/suction channel opening of Pentax gastrointestinal endoscope.
2. This biopsy valve does not accommodate a luer-lock syringe without an irrigating adaptor (part #00711131; sold separately).

00711137 BioShield® irrigator

1. Securely place the biopsy valve with irrigation line onto the biopsy/suction channel opening of Pentax gastrointestinal endoscope.
2. Close the pinch clamp attached to the irrigation line to prevent “whistling” until attachment of a syringe or auxiliary water pump extension tubing for the purpose of irrigating is desired.
3. Fill luer-lock syringe with institutionally approved irrigation fluid (eg. sterile water).
4. To irrigate, open the pinch clamp on the irrigation line and attach the fluid-filled syringe to the irrigation line; depress syringe plunger until desired irrigation effect is achieved; repeat as necessary. **ALTERNATE METHOD:** open the pinch clamp on the irrigation line; attach part #00711134 BioShield® irrigator extension tubing (180cm) to compatible auxiliary water pump, securing the male end of the tubing to the BioShield® device’s irrigation line. Depress the water pump’s foot pedal until desired irrigation effect is achieved; repeat as necessary.
5. Close the pinch clamp on the irrigation line when not in use for irrigation; this will prevent “whistling.”

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Issued Date: May 2012

Warning:

An issued or revision date for these instructions is included for the user’s information. In the event that two years have elapsed between this date and product use, the user should contact US Endoscopy to determine if additional information is available.

Listening...and delivering solutions®, the US Endoscopy listening...and delivering solutions design, and all marks denoted with ® or ™ are registered with the U.S. Patent and Trademark Office, or trademarks owned by US Endoscopy.

Olympus® is a registered trademark of Olympus Optical Co., Ltd.

Pentax® is a registered trademark of Asahi Kogaku Kogyo Kabushiki Kaisha.

Fujinon® is a registered trademark of Fuji Film Co., Ltd.

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Listening...and delivering solutions®, le logo US Endoscopy listening...and delivering solutions, ainsi que toutes les marques portant le signe ® ou ™ sont déposées auprès de l'U.S. Patent and Trademark Office ou sont des marques commerciales détenues par US Endoscopy.
Listening...and delivering solutions®, das US Endoscopy „listening...and delivering solutions“-Design und alle mit ® oder ™ bezeichneten Marken sind Marken, die beim U.S. Patent and Trademark Office (US-Patent- und Markenamt) eingetragen wurden, oder Marken, die sich im Besitz von US Endoscopy befinden.
Listening...and delivering solutions®, il design dello slogan listening...and delivering solutions di US Endoscopy e tutti i marchi contrassegnati da ® o ™ sono stati registrati presso l'US Patent and Trademark Office degli Stati Uniti o sono marchi di fabbrica di proprietà di US Endoscopy.
Listening...and delivering solutions®, o design US Endoscopy listening...and delivering solutions e todas as marcas assinaladas com ® ou ™ estão registadas no Departamento de Registos de Patentes dos EUA (*U.S. Patent and Trademark Office*) ou são marcas comerciais detidas pela US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions design og alle varemærker markeret med ® eller ™ er registreret i USA's patentregister (U.S. Patent and Trademark Office) eller er varemærker, der tilhører US Endoscopy.
Listening...and delivering solutions®, US Endoscopy listening...and delivering solutions 디자인, 그리고 ® 또는 ™로 표시된 모든 상표는 미국 특허청(U.S. Patent and Trademark Office)에 등록되어 있거나 US Endoscopy가 소유하고 있는 상표입니다.